

C-PULSE Pulp tester



Feature

1. Preset speed mode (high-mid-low speed).
2. A gentle, pulsed stimulus begins to increase at a rate
3. Specifically designed for patient comfort.
4. If the patient indicates perception, simply release the button. The stimulus stops immediately, but the numbers remain frozen on the screen for about 3 minutes.
5. Convenient to operate
6. Maximum stimulus-- 80. Between 0-40, the patient feels ache and anesthesia, alive teeth nerve
7. Between 40-80, with above-mentioned reaction, part of teeth nerve dead; 80, no above-mentioned reaction, dead teeth nerve
8. The device turns itself off in three minutes after operation (No wasted batteries.)



Pain threshold graph of the ramped stimulus.



Pulsed stimulus is perceived long before it causes pain.